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Instructions for use and care of uvex medicare comfortable customised inserts*

*Custom-made insole for use in uvex safety shoes

Insertion and use

Your insert is an aid that has been custom made for you and is to be used in combination with your uvex safety shoe. Place your uvex medicare comfortable customised inserts in your uvex safety shoes after you have removed any existing insoles. Your feet may need time to get used to the inserts – we therefore recommend gradually increasing the length of time you wear them and always wearing socks or tights. Your inserts are considered as Class I medical devices and therefore all materials used have been clinically assessed and are deemed safe.

Cleaning and care

We recommend taking the inserts out of the shoe after use to allow them to air and dry. Your inserts can be rubbed clean using a damp cloth and you can also add a conventional disinfectant or soap solution. They are, however, not suitable for cleaning in the washing machine.

The hygiene and service life of the product can be improved through regular care and drying in a well-ventilated area.

Allow the inserts to dry slowly and do not expose them to any heat sources, such as heaters, shoe dryers etc.

Please ensure that the inserts are fully dried before using them again.

The comfortable customised inserts must not be exposed to temperatures over 50°C.

Service life and storage duration

The maximum service life of the inserts is up to 12 months depending on the intensity of use and wear. The maximum storage duration is three years when stored correctly in a dry place without external influences or temperature fluctuations. Stop using the inserts if there are obvious signs of wear, such as cracked or flaking material, or if they are severely misshapen.

Check the function, support and cushioning of your inserts on a regular basis.

Regulations

Inserts are orthopaedic aids as set out by the Medical Devices Act (MPG) and meet the requirements of EU Regulation 2017/745, the Medical Device Regulation (MDR).

Your uvex medicare comfortable customised insert also complies with EU Regulation 2016/425 and has been tested and certified for use in uvex safety shoes.

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